



**Fort Lauderdale
Police Department**

**Emergency: 911
Non-Emergency:
954-764-HELP**

We have partnered with our local establishments to prevent sexual assault. Use the test pad below to check your drink for hidden drugs.

*This test is designed to detect
Gamma Hydroxybutyrate (GHB)
and Ketamine in realistic concentrations,
3.0g/250ml and 1.0g/250ml respectively.*



**DARKER BLUE =
TEST POSITIVE**
**NO CHANGE =
TEST NEGATIVE**



1. Place a drop of your beverage onto both spots of one test.
2. Smear each gently, wait until dry.
3. If either spot turns to a darker blue color see back for further instructions.

Bar Watch Coaster Front